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

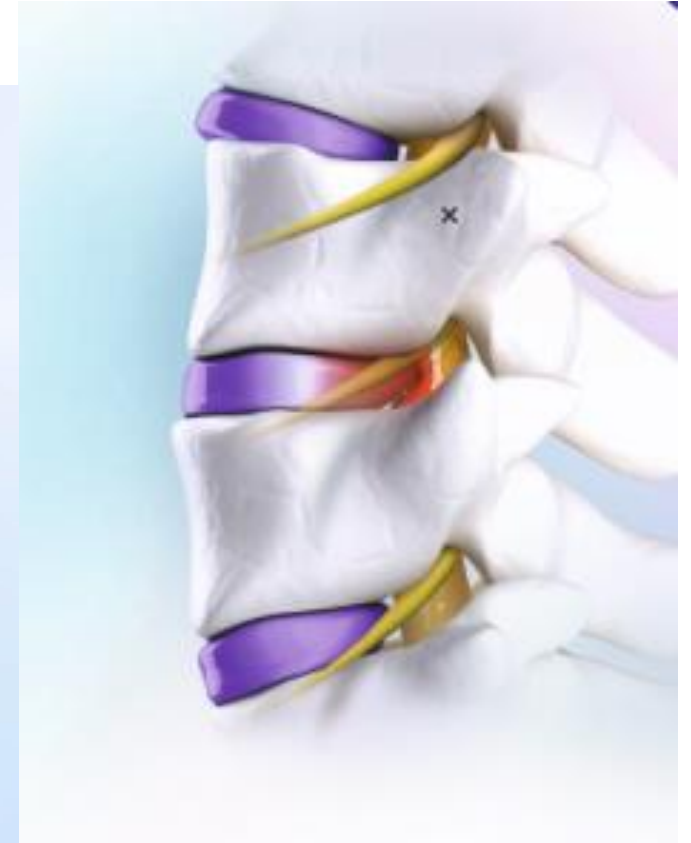
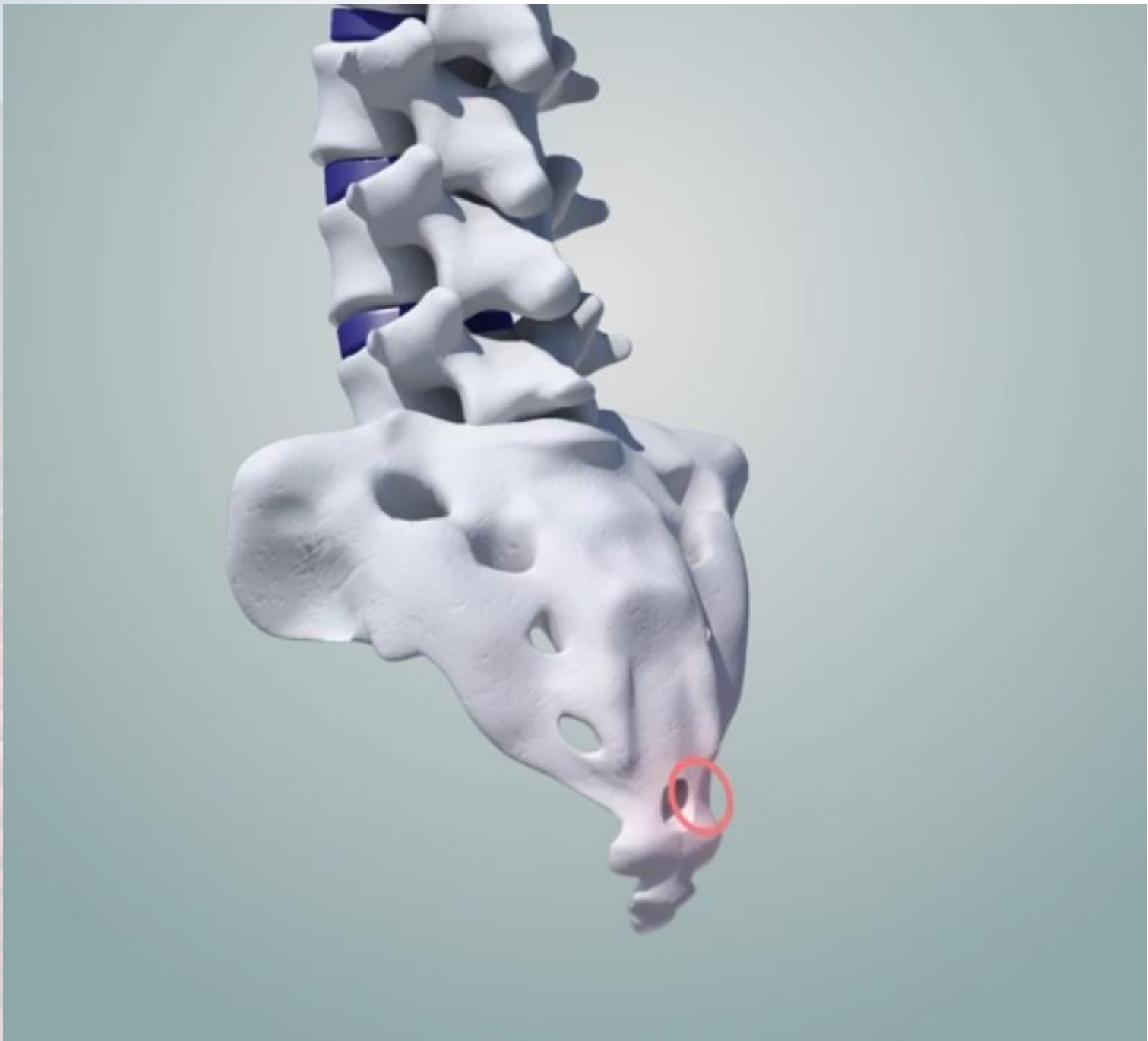
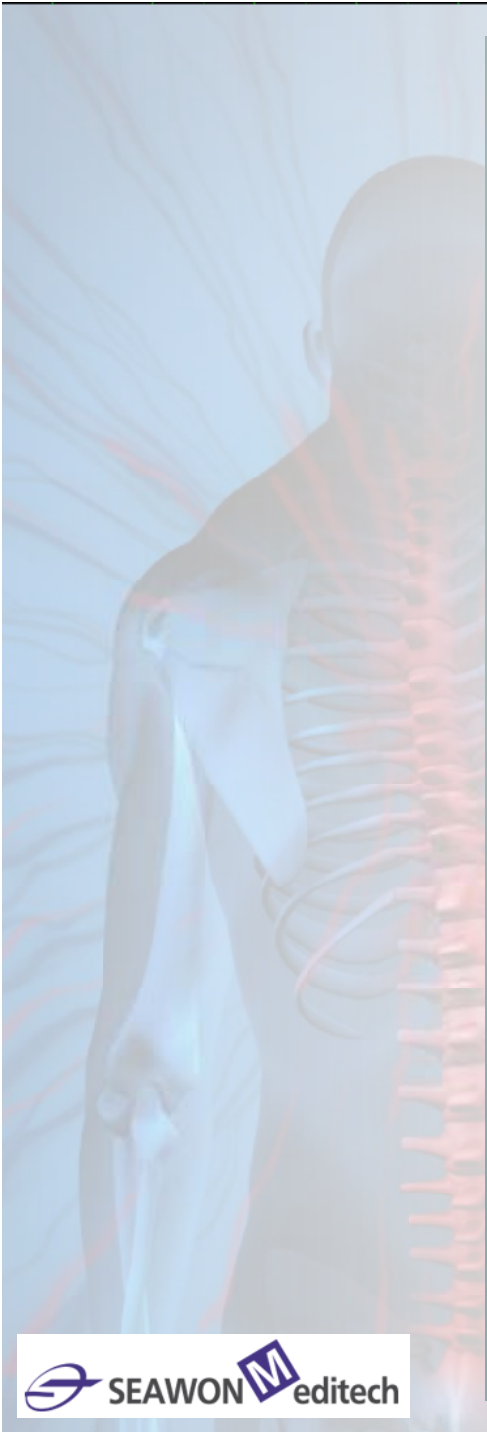
STEP 2

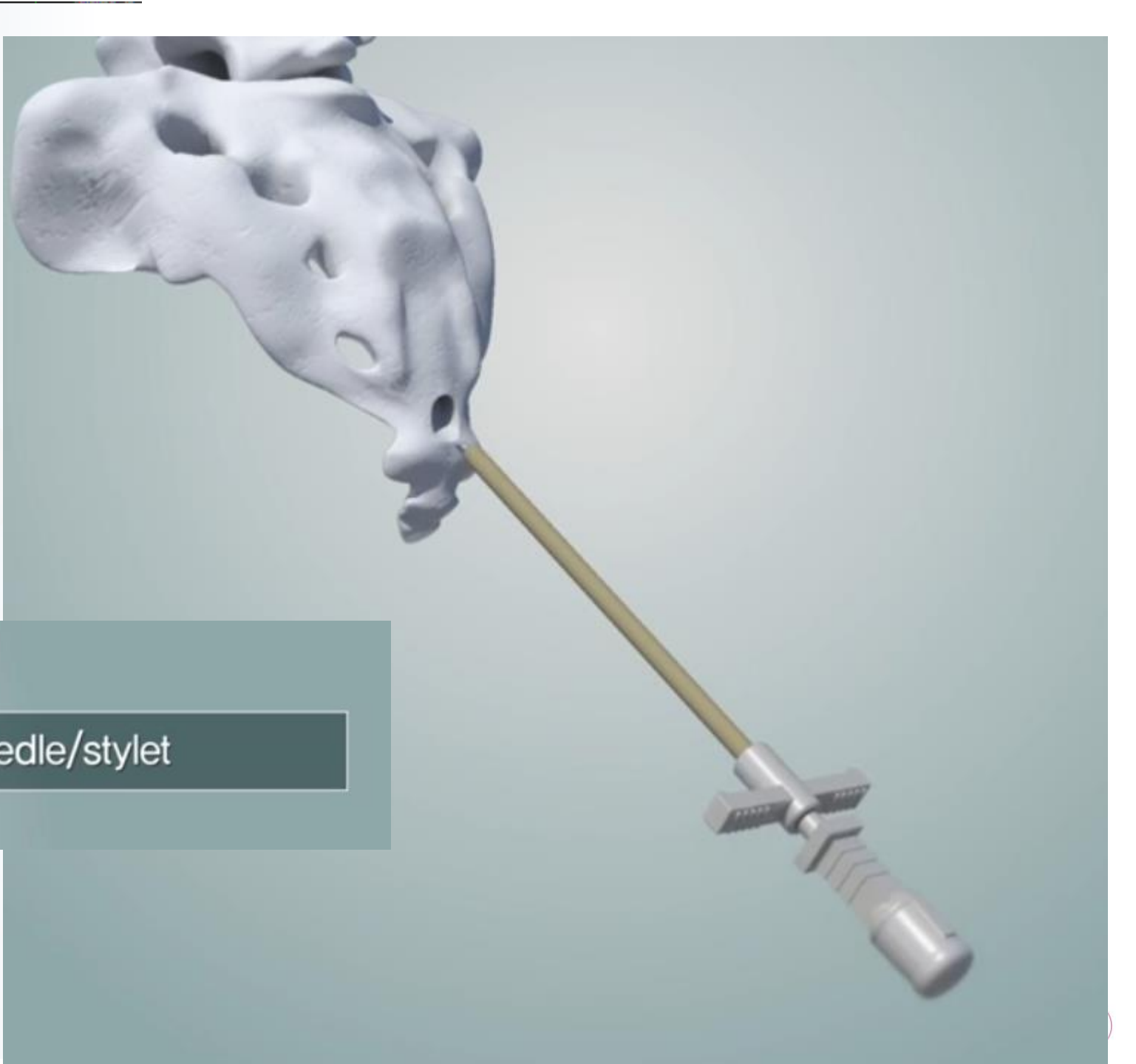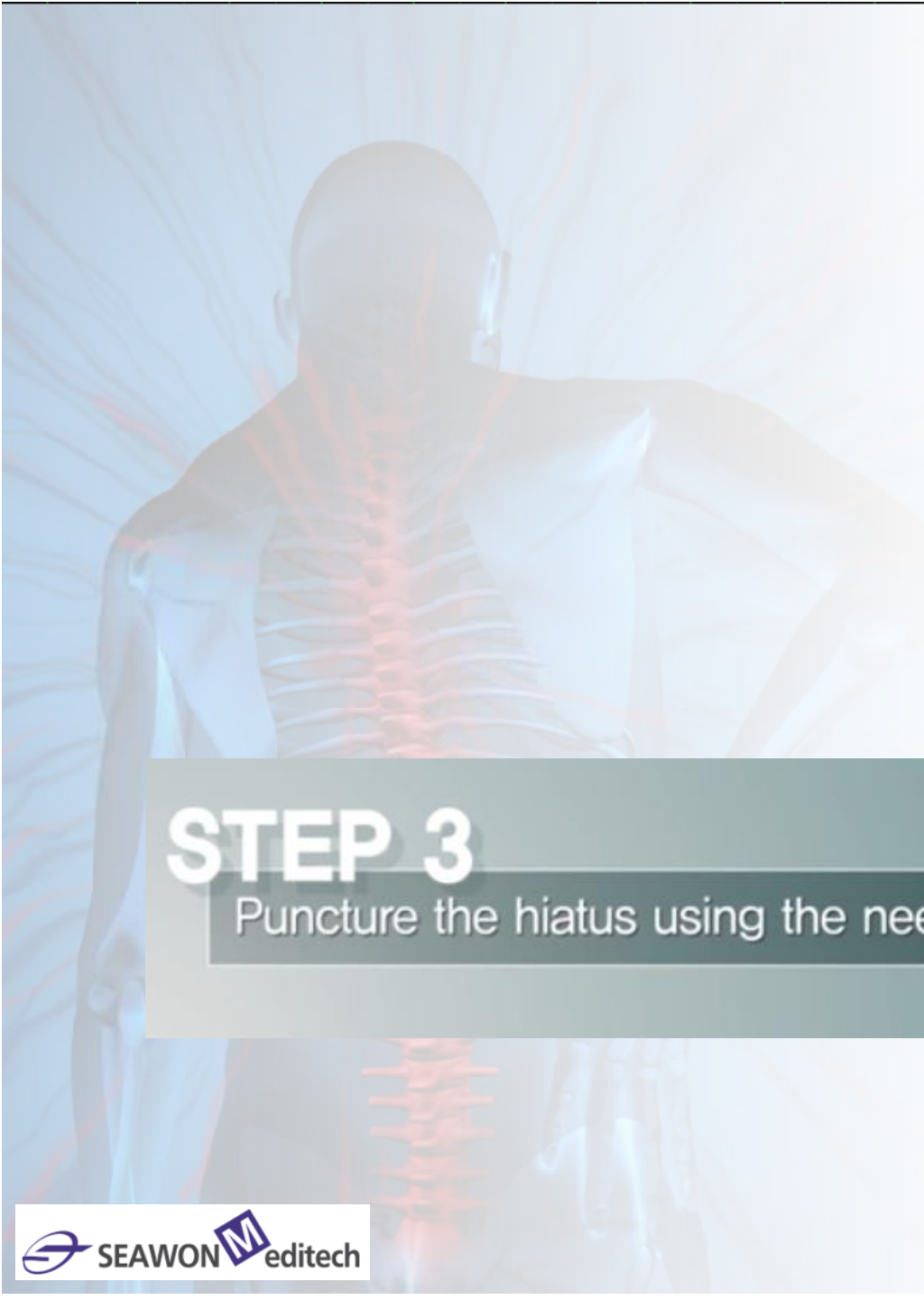Inject lidocaine to anesthetize the patient for local anesthesia inside hiatus.

STEP 3

Puncture the hiatus using the needle/styler

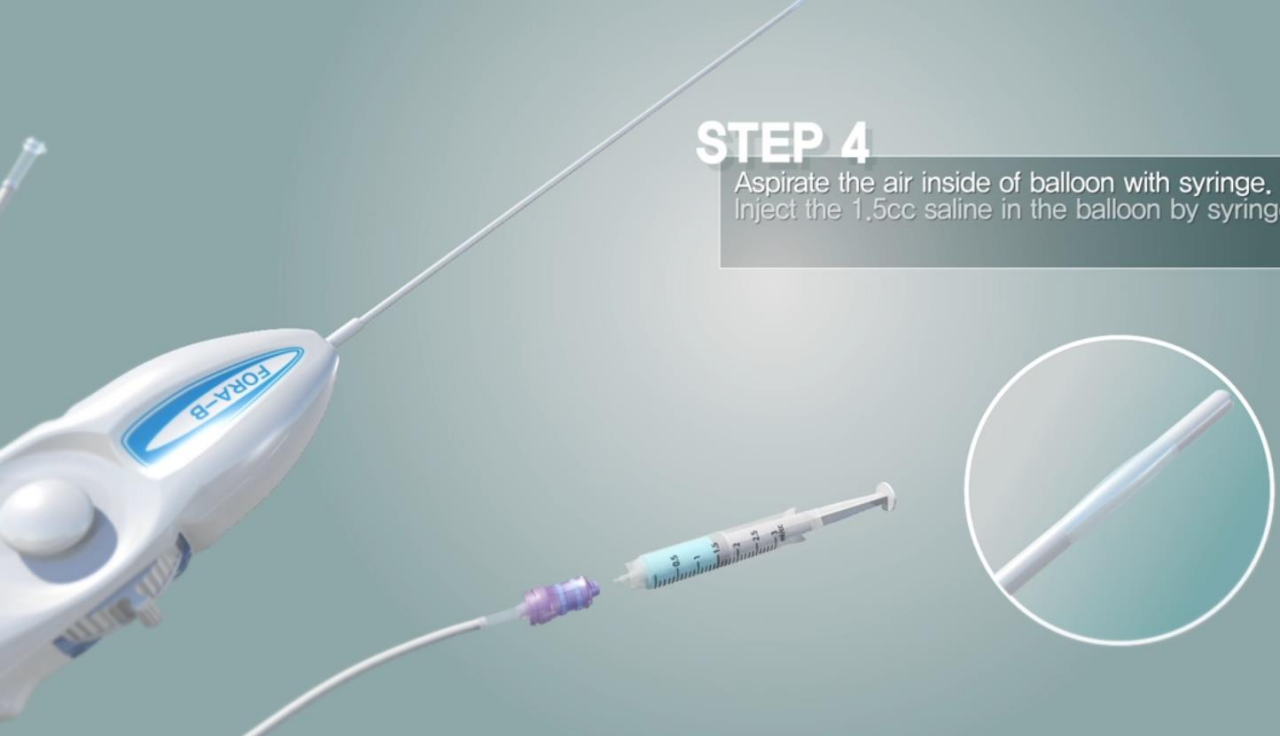
STEP 4

Aspirate the air inside of balloon with syringe. (3 times)


STEP 4

Aspirate the air inside of balloon with syringe. (3 times)
Inject the 1,5cc saline in the balloon by syringe, to check

STEP 5

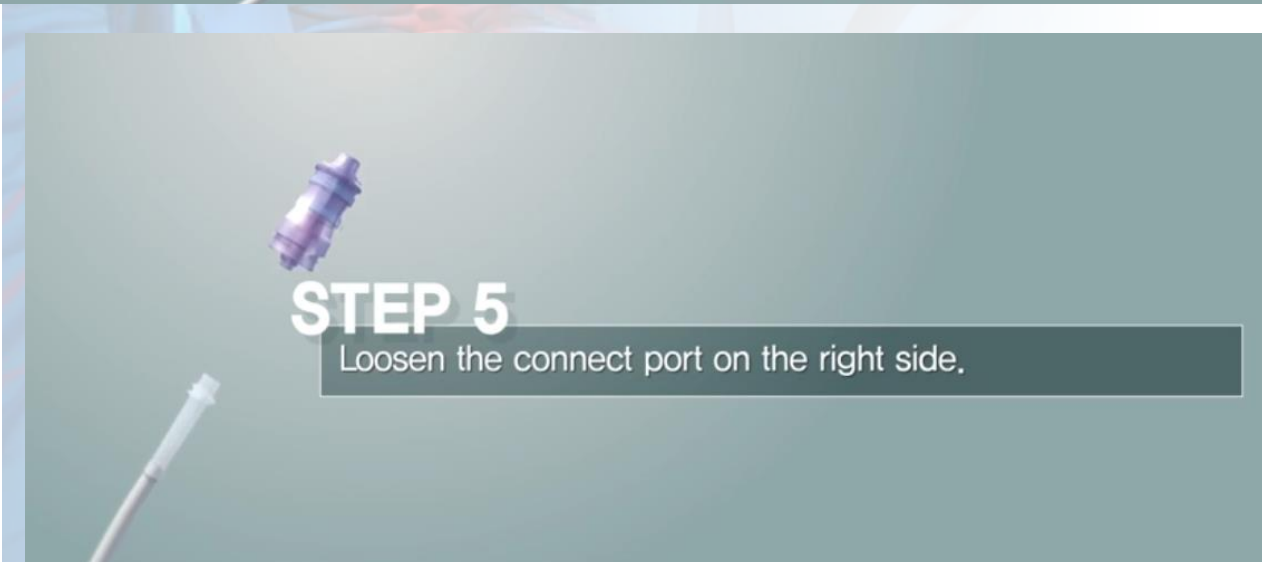Loosen the connect port on the right side.


STEP 4


Aspirate the air inside of balloon with syringe. (3 times)
Inject the 1,5cc saline in the balloon by syringe, to check


STEP 5

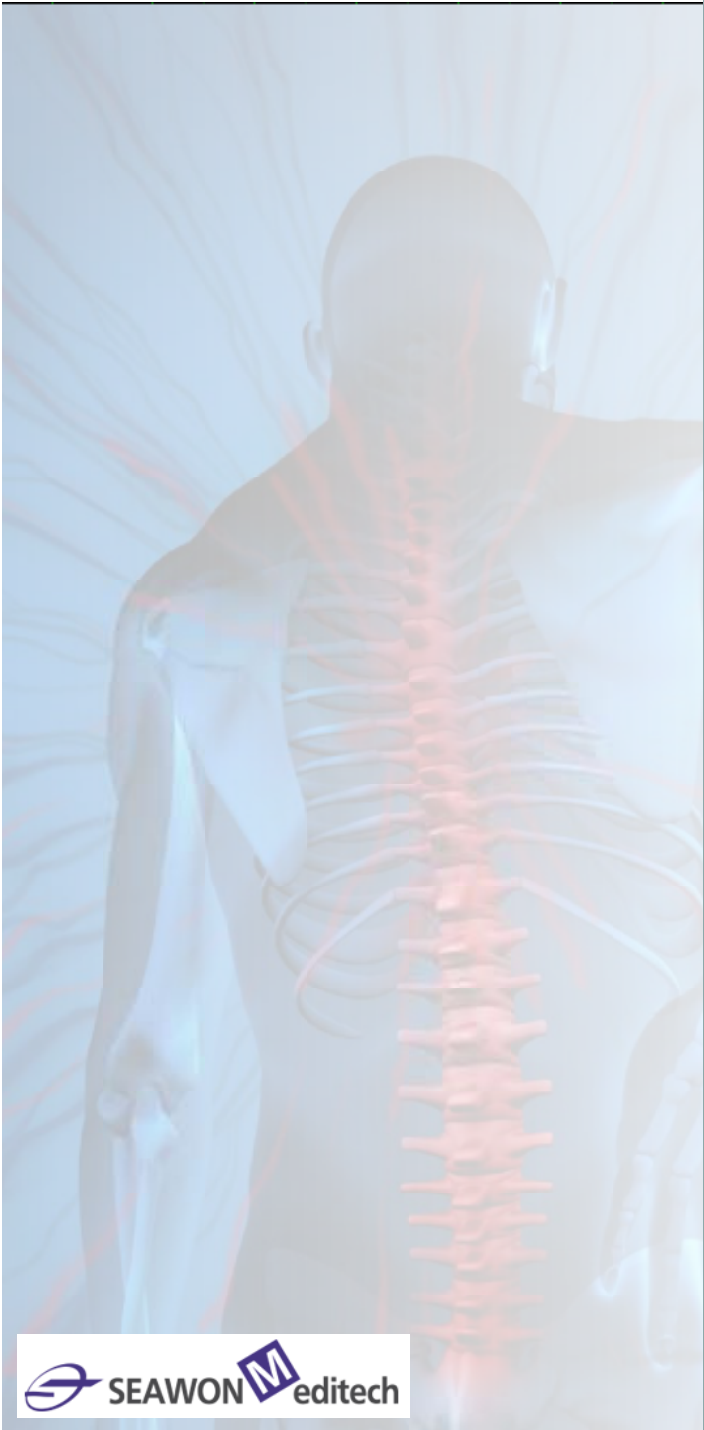Loosen the connect port on the right side.


ST

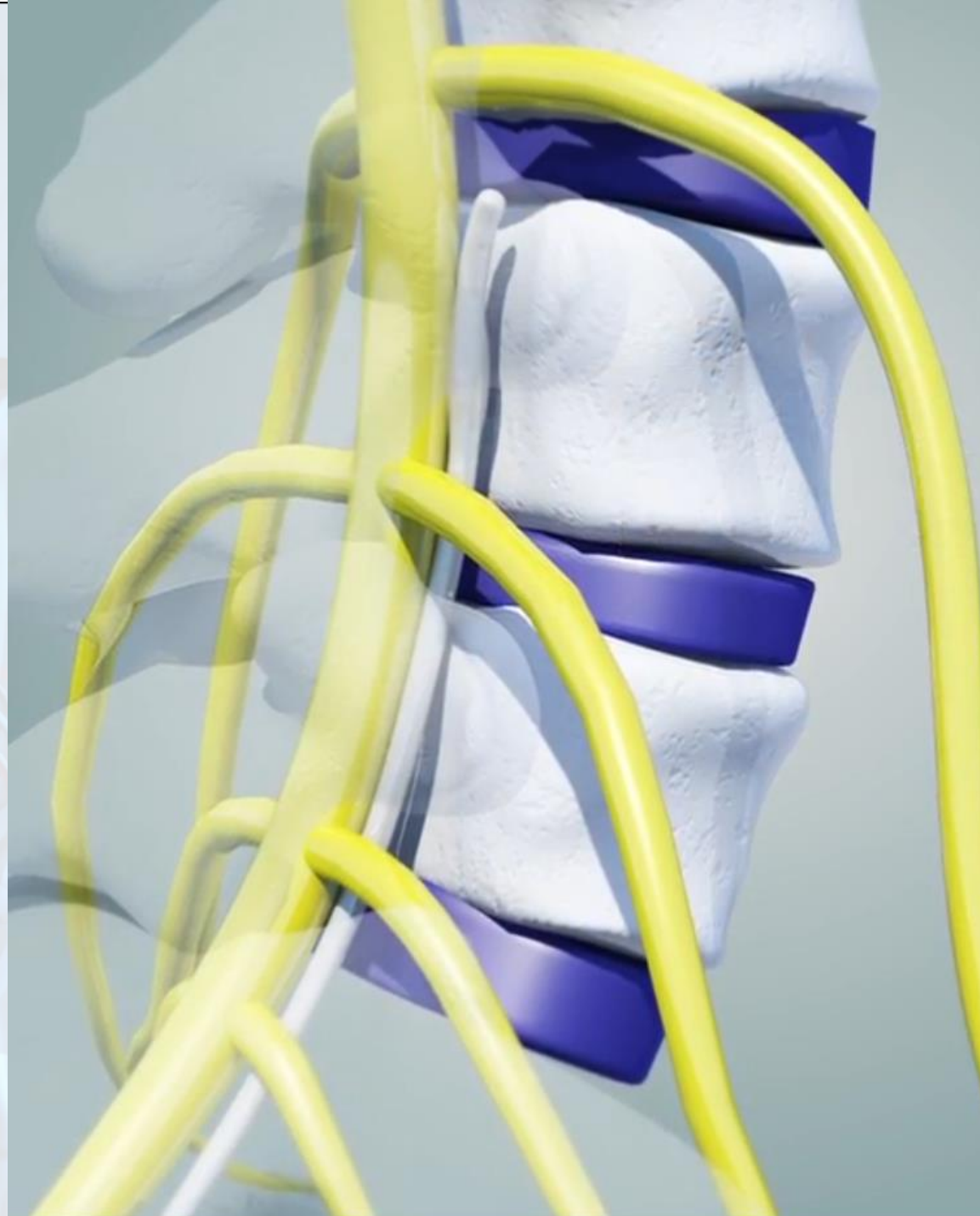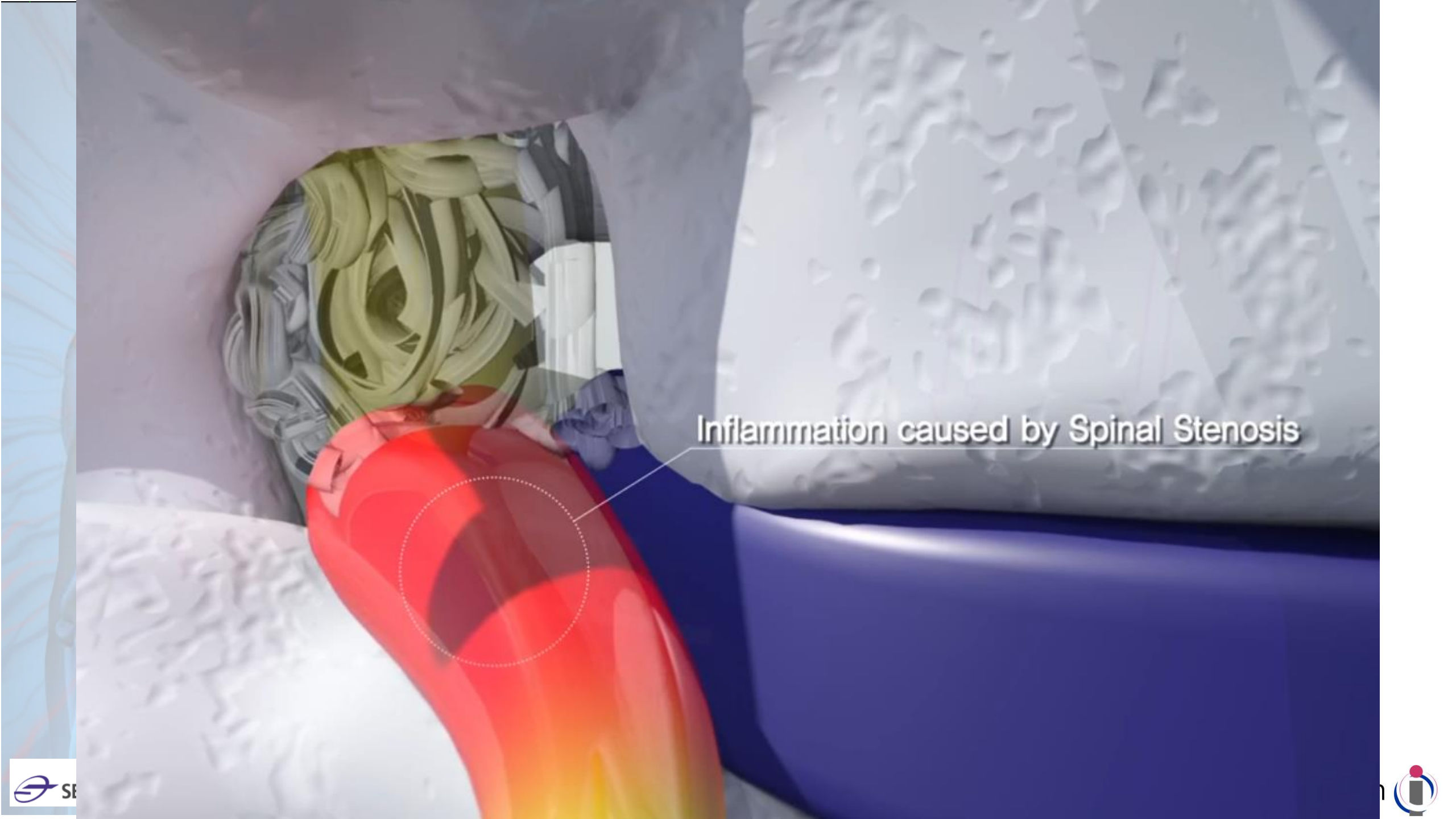A 3D anatomical model of a lumbar vertebra. The vertebral body is colored with a red-to-yellow gradient, indicating inflammation. A dashed white circle highlights a specific area on the anterior surface of the vertebral body. A white line connects this circle to a text label. The surrounding bony structures are shown in a light gray, semi-transparent style.

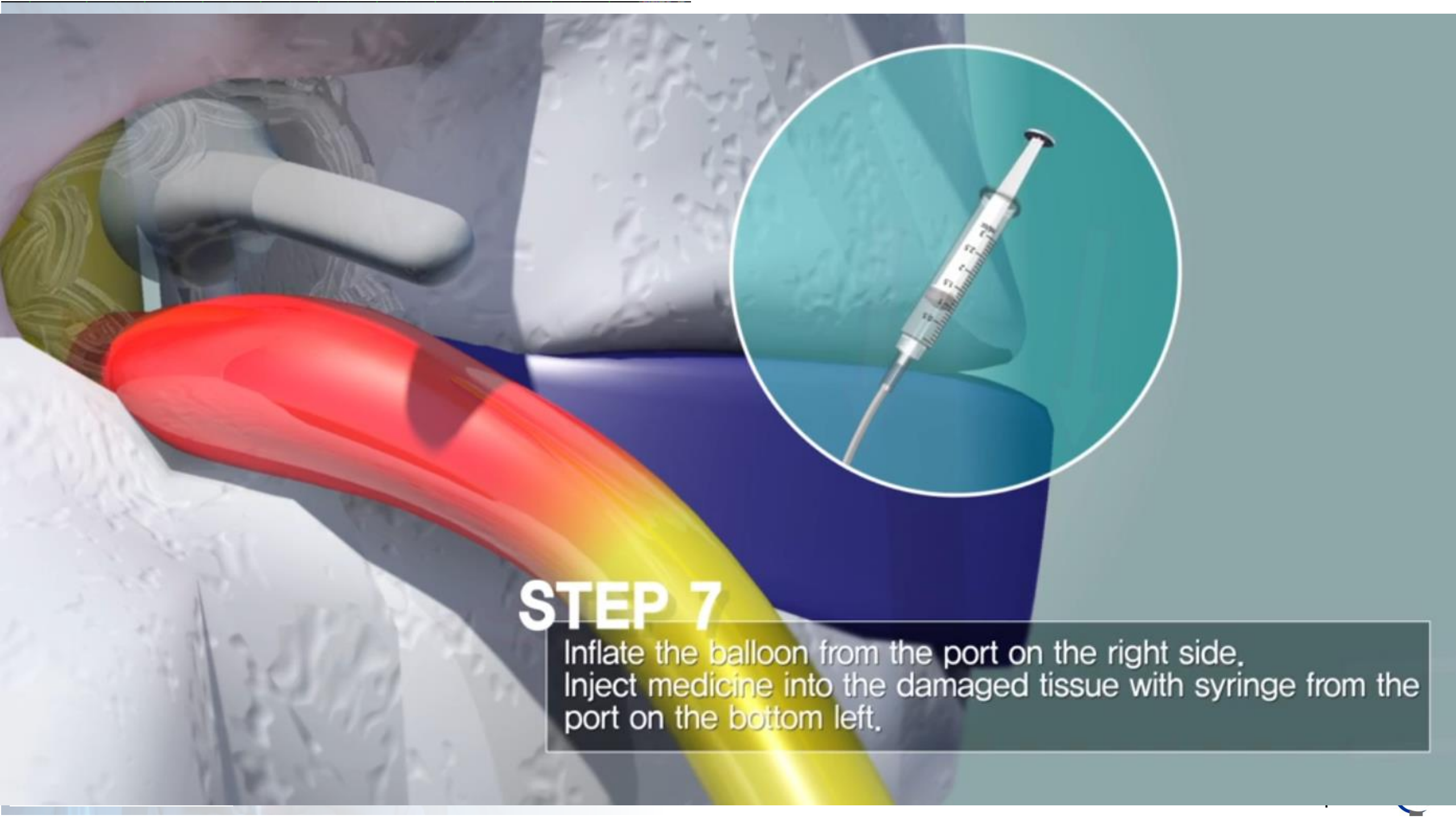Inflammation caused by Spinal Stenosis

STEP 7

Inflate the balloon from the port on the right side.
Inject medicine into the damaged tissue with syringe from the
port on the bottom left.